


Valplast flexible partial dentures now available!

The esthetic, metal-free alternative to cast partial frames.

Also available: Valplast with a Wironium subframe when extra strength is needed in extended edentulous areas.

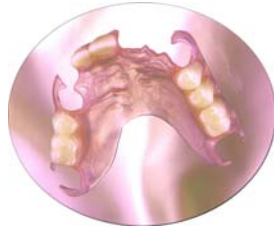

Valplast Dentist info and Patient care instructions-

Initial Insertion: The Valplast partial should be immersed in very hot (near boiling) water for one minute prior to insertion. Allow the surface to cool to the point that the patient will tolerate it easily and then insert the prosthesis. The heat exposure will assure a very smooth initial insertion, often relieving any irritation the patient may feel the first time a partial is inserted.

Adjustments: If you observe that a clasp needs to be tightened or loosened, immerse the section to be adjusted in very hot water for one minute. As soon as it is removed from the water, bend the clasp slightly in the direction that achieves the desired fit. Immediately insert the partial in the patient's mouth and allow it to conform for a short time before making any further adjustments (if needed).

Trimming and chairside adjustments: If adjustment or trimming is required, we recommend using a #13 Green Mounted Stone. A Brown Rubber Wheel should then be used to smooth over the polishable surface after any adjustments are made. The rubber wheel can also be used for minor delicate reduction or relieving.

Patient Care: Valplast appliances should be kept in water when they are not being worn. To keep Valplast appliances hygienic and attractive, they should be immersed overnight in a good cleanser at least every other night. *Bleaches and cleansers using bleach should not be used.* We offer the Val-Clean concentrated denture cleaner for \$12 per box, which provides a 6 month supply. This concentrated cleaner allows for a twenty minute soak daily to keep the appliance clean. *Give us a call to order the Val-Clean cleaner.*

Call today for quick pick up of your next Valplast case!

A.D.A. codes for flexible partials are-
Upper- 05225
Lower- 05226



**We need 5 days in lab to complete Valplast.*

Gnathodontics, Ltd.

303-424-9515 * 800-234-9515